

PRECISION 3D RESINS

KeyDenture® Base



Cleaning and Care Instructions

REV 0 03/2025

Welcome to your KeyDenture® experience! This denture has been customized for you and designed for comfort. Wear your denture as prescribed by your dental professional and follow your daily routine of brushing and flossing after each meal. Keep up with regular dental appointments and check the fit and condition of your appliance during each visit. If you have any new dental work done, confirm the fit of this appliance with your dental professional. If you experience discomfort or pressure, please contact your dental professional for follow-up.

Cleaning and Maintaining your KeyDenture Base

- Remove and rinse dentures after eating. Run water over your dentures to remove food debris and loose particles.
- Clean your mouth after removing your dentures. Use a soft-bristled tooth brush on natural teeth, tongue and palate.
- Scrub your dentures at least once a day. Gently scrub your dentures using a denture cleaner, mild soap or dishwashing liquid and a denture brush or soft toothbrush to remove food, plaque and other deposits. Avoid stiff-bristled brushes, as they can damage dentures. Whitening toothpastes can also be abrasive on the dentures and should be avoided.
- Place the dentures in water or a mild denture-soaking solution overnight. Check with your dentist about properly storing your dentures.
- Rinse dentures before putting them back in your mouth, especially if using a denture-soaking solution. These solutions can contain harmful chemicals that cause vomiting, pain or burns if swallowed.
- Schedule regular dental checkups. Your dentist will advise you about how often to visit to have your dentures examined and professionally cleaned. Your dentist can help ensure a proper fit to prevent slippage and discomfort. See your dentist promptly if your dentures become loose.

KEYPRINT.KEYSTONEINDUSTRIES.COM

480 South Democrat Road Gibbstown, NJ 08027 USA
Stockholzstr. 11, D-78224 Singen, Germany



**PLEASE VISIT OUR
KEYPRINT WEBSITE
TO LEARN MORE!**